

LIMITED LIFETIME WARRANTY

NOKA warrants the original purchaser of this product against any defects in material or workmanship under normal use. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOKA warrants its products for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame: Limited Lifetime Warranty

Non-Metal/Metal parts: One year Limited Warranty

This warranty does not cover failures due to user negligence, misuse of the product, improper operations and unauthorized modifications. The warranty does not extend to the normal wear and tear of the non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

For warranty service, please contact the dealer from whom you purchased your product. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

DO NOT return any products to our office without prior consent.

The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein. NOKA shall not be liable for any consequential or incidental damages whatsoever.

www.nokamedical.com

Made in China



OVERBED TABLE STANDARD



Important Safety Notice

- 1: Assemble Manually: Use hand tools only; avoid power tools to prevent over-tightening and damage.
- 2: Regular Inspections: Routinely check for cracks, loose parts, or instability.
- 3: Weight Limit: Do not exceed the maximum load of 22 lbs.

INSTRUCTIONS

READ INSTRUCTIONS CAREFULLY BEFORE INSTALLING.

Part list


- ▶ Tabletop x1
- ▶ Base x1
- ▶ Bolt plate x1
- ▶ Bolts x3
- ▶ Column assembly x1
- ▶ Short screws x6
- ▶ Wheels with lock x2
- ▶ Wheels without lock x2
- ▶ Small hex key x1
- ▶ Big hex key x1
- ▶ Small spanner x1
- ▶ Big spanner x1

Assembly

- ▶ Place the tabletop on a flat surface with the screw holes facing up. Align the column assembly with the holes, then use the small hex key to install and fully tighten all six short screws through the top bracket into the tabletop. (Picture 1,2)



1

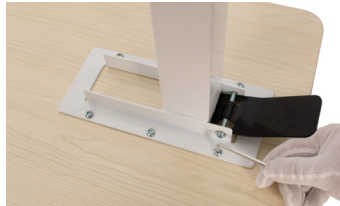

2

- ▶ Position the base onto the column, then install the three bolts with washers and spacers, and tighten them securely using the big hex key and small spanner. (Picture 3,4)



3



4

- ▶ Insert each wheel fully into the base holes. Use the big spanner to install the locking wheels near the column and the non-locking wheels on the opposite side. (Picture 5)



5

HEIGHT ADJUSTMENT

To raise

- ▶ Pull up from the bottom of the table top until you reach your desired height



To lower

- ▶ Pull up on the lever and pull up from the bottom of the table top until you reach your desired height

WARNING

- ▶ Do not over tighten the bolts, screws or use any power-tolls when assembling this product.
- ▶ Inspect the overbed table for damage or loose hardware before each use. Do not use the table if it is damaged or unstable.
- ▶ Do not modify the table in any way.
- ▶ The overbed table is designed for indoor use only.
- ▶ Keep any weight on the overbed table evenly distributed over the surface.
- ▶ Do not exceed 22 lbs, as overloading may cause it to brake or serious injury.
- ▶ Only use the overbed table for its intended use.
- ⚠ The overbed table should not be used as a sleep surface; as a walker or mobility aid, to prevent or brace a person's fall; or to help transport persons or items.