



# Shower Chair with Arms & U-Shaped Hygienic Cut-out User Guide

ITEM: #1011A-S



## Important Safety Notice

Beware of slipping; avoid falling. Do not stand or squat on the chair.

Do not leave unattended while using the chair.

Max Weight Capacity of **300 lbs**


Read this user guide before operating or using your NOKA Shower Chair.

User guide and video available online at [www.nokamedical.com](http://www.nokamedical.com)


# INSTRUCTIONS

READ INSTRUCTIONS CAREFULLY BEFORE INSTALLING.


## Components

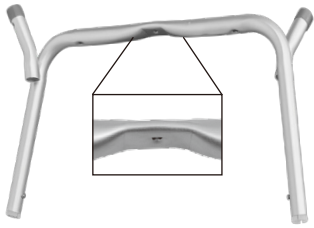


Main Frame Tube 1

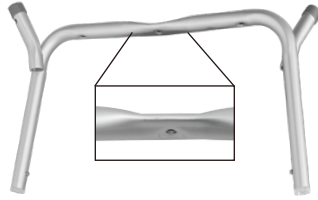

Main Frame Tube 2

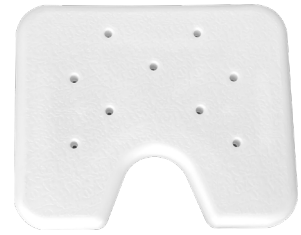

Seat

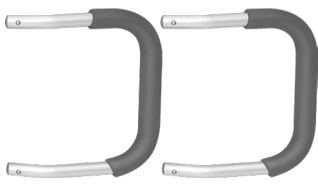


Armrest



Leg





Screws

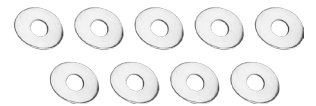


Arc-shaped Washer

- ▶ Install the first bracket, and place the main frame tube with the indentation in the middle corresponding to the groove in the bench;
- ▶ Install the second bracket into the corresponding groove.

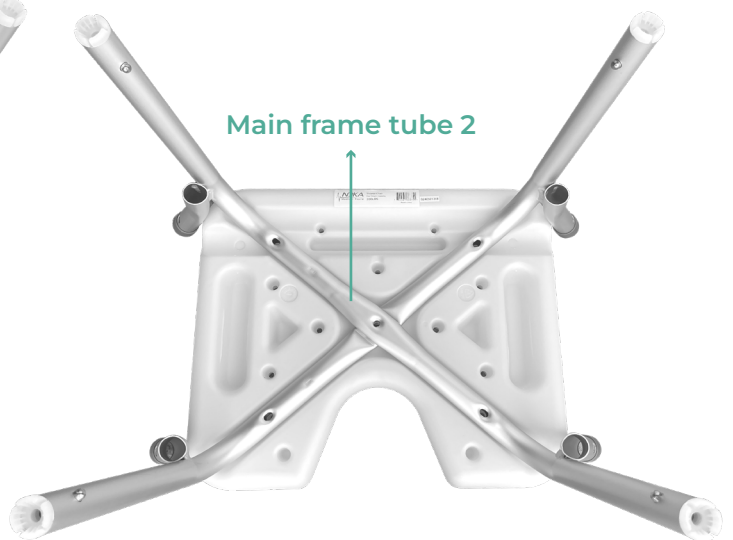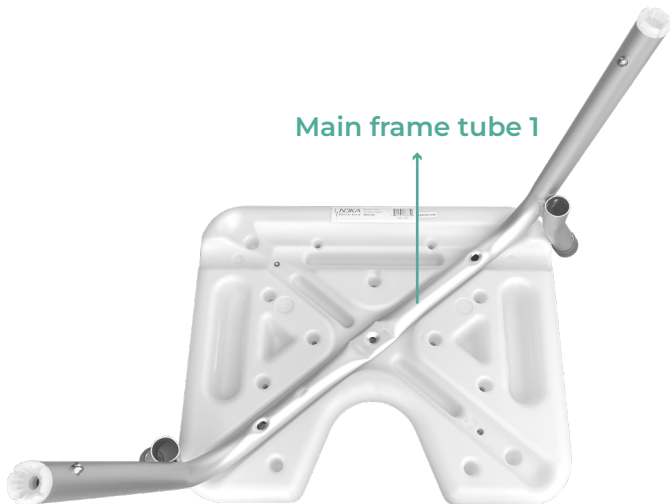

- ▶ The arc-shaped washer is as shown in the picture, and the correct placement direction is that the arc-shaped part fits the aluminum tube.
- ▶ Place the arc-shaped washer and the short screw in turn at the intersection of the backrest bracket and tighten down.
- ▶ The arrow position of the seat plate of the base is the direction of installing the backrest tube, put the backrest tube in the corresponding position, put 4 arc washers and 4 long screws in turn to tighten.

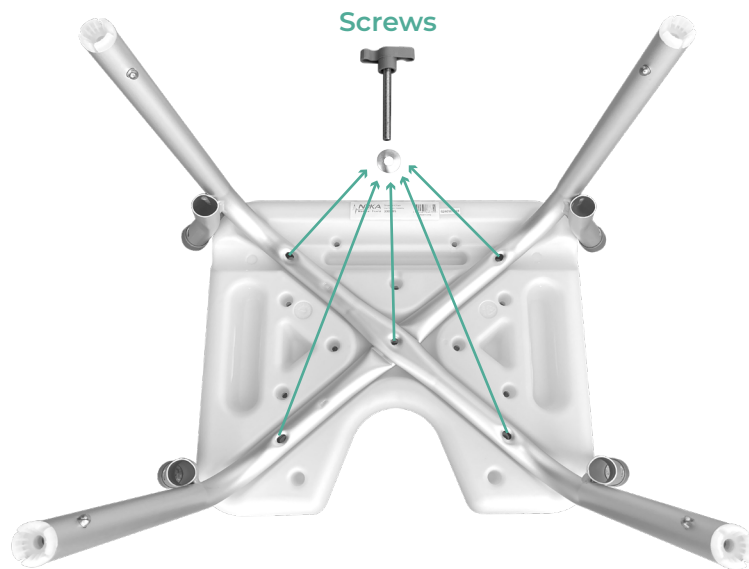

- Install 4 moving legs with anti-skid pads. Ensure the brass push-pin “snaps” through the hole. Make sure the legs are at the same height.
- Insert the arm into the arm sockets at the sides of the seat. Ensure the brass push-pin “snaps” through the hole. Make sure the arm arches outwards.



## HEIGHT ADJUSTMENT

- Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
- Be sure legs are locked into place by pushing down on or by pulling legs.



## WARNING

- ▶ Make certain that the height adjusting brass pushpins fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
  - ▶ Ensure ALL screws, nuts and/or bolts are tightened.
  - ▶ Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
  - ▶ All four leg extensions with rubber tips MUST touch the floor simultaneously.
  - ▶ When used in a tub, make certain the rubber tips are not touching the sides of the tub wall.
- ⚠ DO NOT stand on chair.

## LIMITED LIFETIME WARRANTY

NOKA warrants the original purchaser of this product against any defects in material or workmanship under normal use. The warranty does not extend to any subsequent purchaser or owner, The warranty is void upon any sale or transfer of ownership or use by other person. NOKA warrants its products for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

**Metal Frame: Limited Lifetime Warranty**

**Non-Metal/Metal parts: One year Limited Warranty**

This warranty does not cover failures due to user negligence, misuse of the product, improper operations and unauthorized modifications. The warranty does not extend to the normal wear and tear of the non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

For warranty service, please contact the dealer from whom you purchased your product. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

DO NOT return any products to our office without prior consent.

The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein. NOKA shall not be liable for any consequential or incidental damages whatsoever.

[WWW.NOKAMEDICAL.COM](http://WWW.NOKAMEDICAL.COM)