

LIMITED LIFETIME WARRANTY

NOKA warrants the original purchaser of this product against any defects in material or workmanship under normal use. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOKA warrants its products for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame: Limited Lifetime Warranty

Non-Metal/Metal parts: One year Limited Warranty

This warranty does not cover failures due to user negligence, misuse of the product, improper operations and unauthorized modifications. The warranty does not extend to the normal wear and tear of the non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

For warranty service, please contact the dealer from whom you purchased your product. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

DO NOT return any products to our office without prior consent.

The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein. NOKA shall not be liable for any consequential or incidental damages whatsoever.

www.nokamedical.com

Made in China



BATHROOM SAFETY



Important Safety Notice

Beware of slipping; avoid falling.

Do not stand or squat on the chair.

Do not leave unattended while using the chair.

Max Weight Capacity of 300 lbs


INSTRUCTIONS

READ INSTRUCTIONS CAREFULLY BEFORE INSTALLING.

- Lay the chair base with the seat surface facing down and the leg sockets facing up.
- Depress the brass push-pin and insert the leg into the socket until the brass push-pin “snaps” through the hole.
- Repeat step 2 with the other three legs.
- Turn chair upright.
- (For the model with backrest) Insert the two L-shaped back support tubes into the tubes on the chair back. Ensure that the brass push-pin “snaps” through the hole.
- (For the model with armrest) Insert the arm into the arm sockets at the sides of the seat. Ensure the brass push-pin “snaps” through the hole. Make sure the arm arches outwards. Repeat with other arm.

HEIGHT ADJUSTMENT

- Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes (see Figure 1).
- Be sure legs are locked into place by pushing down on or by pulling legs.

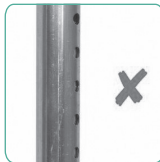

WARNING

- Make certain that the height adjusting brass pushpins fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
- Ensure ALL screws, nuts and/or bolts are tightened.
- Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
- All four leg extensions with rubber tips **MUST** touch the floor simultaneously.
- When used in a tub, make certain the rubber tips are not touching the sides of the tub wall.
- ⚠ DO NOT stand on chair.