



# Aluminum Rollator User Guide

ITEM: #007CH, 007CB, 007RD



## Important Safety Notice

Do not have anyone push you while you are seated on the rollator. This is a walking aid only and is not to be used as a transportation device.

Max Weight Capacity of **300 lbs**

Read this user guide before operating or using your NOKA rollator.

User guide and video available online at [www.nokamedical.com](http://www.nokamedical.com)

# INSTRUCTIONS

Do not adjust or operate rollator without carefully reading all instructions in this manual.


- ▶ First remove all contents from carton: backrest, handle with tube, rollator frame and basket. Open the frame. Ensure that the folding hinge is locked into place. Slide the backrest into frame brackets making sure snap bottoms protrude through holes.
- ▶ Unscrew height adjustment knob from the frame and insert handle tube into frame at desired height, lining up holes appropriately. Slide bolts through holes from the inside and screw on tightening knob until secure.

⚠ **DO NOT** exceed weight capacity.

⚠ **DO NOT** self-propel or scoot around while seated.

## Components

- ▶ Rollator Main Frame (①)
- ▶ Two Hand Grips (②)
- ▶ One Backrest (③)
- ▶ Two Handle Adjustment Screws (④)

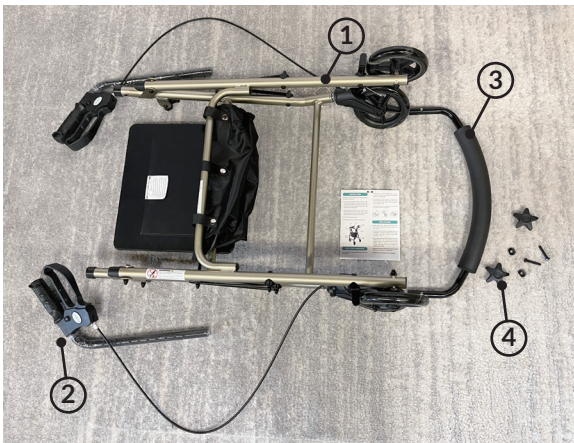




## PRECAUTIONS

- ▶ Please make sure the rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- ▶ Be sure that both knobs are fully tightened and are at equal heights. Test rollator for stability before applying full weight to hand grips. Test the loop locks before operating the rollator.
- ▶ When wheels are locked (see illustrations for locking and releasing locks), lock bar on rear wheels should prevent wheels from turning. If the wheels turn while the brake is engaged, contact your dealer for instructions to adjust the hand brake system. When locks are released, the wheels should turn freely.
- ▶ Always lock loop locks before sitting and do not use seat when unit is on an incline or uneven ground.
- ▶ Replace tires when significantly worn or damaged.

## LOOP LOCK OPERATION

Always engage and disengage wheel locks on both sides simultaneously.

- ▶ To slow down or come to a temporary stop — pull up slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop.
- ▶ To temporarily lock wheels in position — press down on both loop locks with palms until you hear a “click” that will tell you wheels are locked. Always lock the wheels when using the rollator as a seat.
- ▶ To release wheel locks — push up on both loop locks with palms of hand to original position.

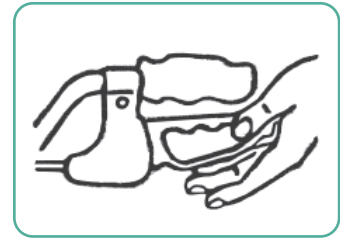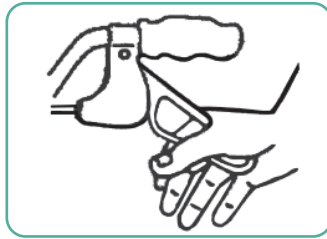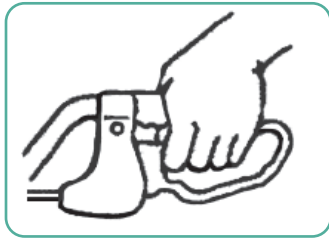

## LIMITED LIFETIME WARRANTY

NOKA warrants the original purchaser of this product against any defects in material or workmanship under normal use. The warranty does not extend to any subsequent purchaser or owner, The warranty is void upon any sale or transfer of ownership or use by other person. NOKA warrants its products for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

**Metal Frame: Limited Lifetime Warranty**

**Non-Metal/Metal parts: One year Limited Warranty**

This warranty does not cover failures due to user negligence, misuse of the product, improper operations and unauthorized modifications. The warranty does not extend to the normal wear and tear of the non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

For warranty service, please contact the dealer from whom you purchased your product. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

DO NOT return any products to our office without prior consent.

The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein. NOKA shall not be liable for any consequential or incidental damages whatsoever.